Stat Profile® pHox® Plus M Controls Auto-Cartridge

Kontroll-Auto-Kassetten, Αυτόματη Κασέτα Διαλυμάτων Ελέγχου, Cartucho automático para controles, Cartouche CQ, Cartuccia automatica controlli, Cartucho automático de control, Kontroller, autokassett

CONTROL 1 2 3

LOT 810678

2010-02

Expected Ranges, Erwartete Bereiche, Αναμενόμενα Εύρη, Rangos esperados, Fourchettes attendues, Intervalli previsti, Gamas previstas, Förväntade områden

Table with 5 columns: Parameter, Unit, CONTROL 1, CONTROL 2, CONTROL 3. Rows include pH, H+, PCO2, PO2, Hct, Na+, K+, Mg++, Ca++, Glu, and Lac.

Product Description: Consists of 3 flexible bags within a cardboard carton. Each bag contains an aqueous quality control material for monitoring the measurement of pH, PCO2, PO2, hematocrit (Hct) Na+, K+, Mg++, Ca+, Glucose (Glu) and Lactate (Lac) for use with Nova Biomedical analyzers ONLY.

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Intended Use: For in vitro diagnostic use for monitoring the performance of Nova Biomedical Stat Profile pHox Plus M Analyzers.

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Methodology: Refer to Stat Profile pHox Plus M Analyzer Instructions For Use Manual.

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Warnings and Cautions: Must be stored at 24-26°C for at least 24 hours before use. DO NOT FREEZE.

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Directions for use: Cartridge must be stored at approximately 24-26°C for at least 24 hours prior to opening.

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Traceability of Standards: Analytes are traced to NIST Standard Reference Materials.

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Expected Ranges: The EXPECTED RANGE for each analyte was determined at Nova Biomedical by using multiple runs of each level of control at 37°C on multiple instruments.

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How to Define and Determine Reference Intervals in the clinical laboratory: approved guideline-second edition, NCCLS C28-A2, Volume 20, Number 13

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EC REP Nova Biomedical UK

C3-5, Evans Business Centre, Deeside Industrial Park, Deeside, Flintshire, UK, CH5 2JZ